


## QUICK REFERENCE GUIDE

OPEN POUCH JUST PRIOR TO COLLECTION.

### PRODUCT STORAGE & COLLECTION

STORE AT ROOM TEMPERATURE

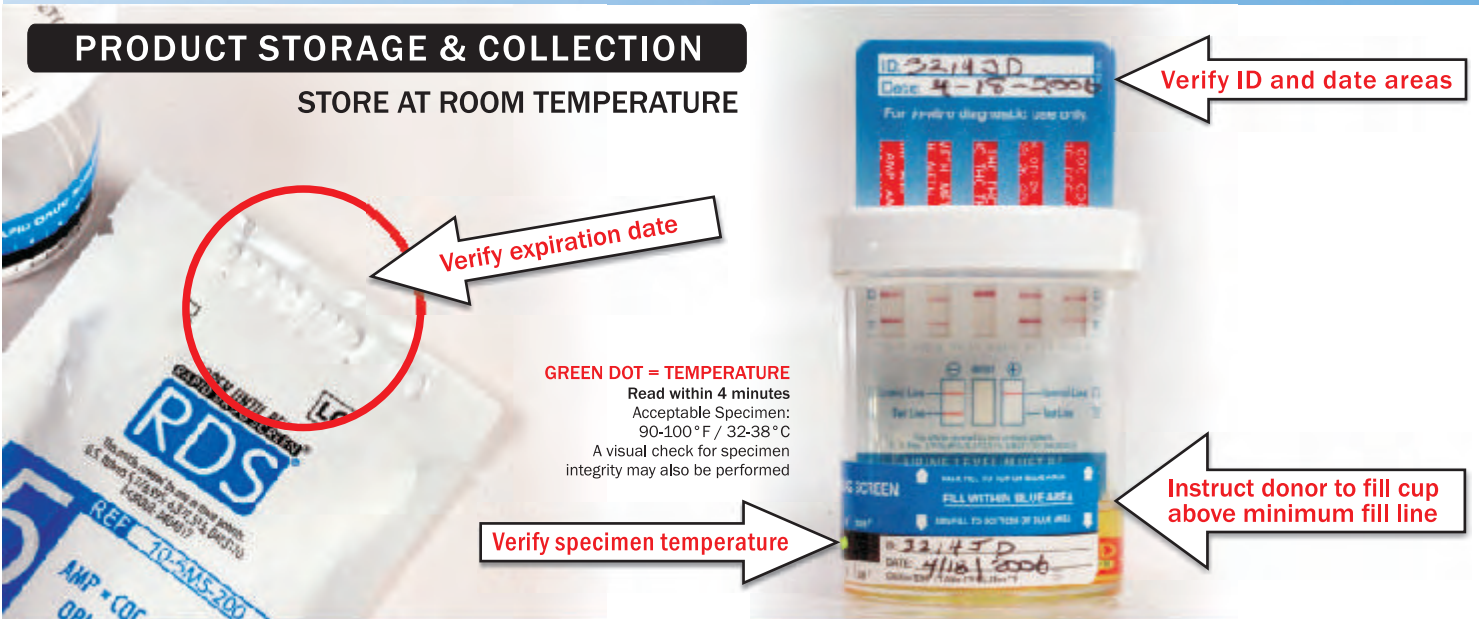

An integrated collection and testing device capable of testing for up to 10 drugs of abuse in urine.


	Cut-off ng/ml	Detection Times (approximate)
<b>AMP</b> amphetamine	1000	2-4 days
<b>BAR</b> barbiturates	300	2-4 days
<b>BZO</b> benzodiazepines	300	Up to 2 weeks
<b>BUP</b> buprenorphine	10	2-3 days
<b>COC</b> cocaine	150 300	1-3 days
<b>MDMA</b> ecstasy	1000	2-4 days
<b>METH</b> methamphetamine	1000	1-2 days
<b>MTD</b> methadone	300	1-4 days
<b>OPI</b> opiates	300 2000	1-3 days
<b>OXY</b> oxycodone	100	1-3 days
<b>PCP</b> phencyclidine	25	3-8 days
<b>PPX</b> propoxyphene	300	1-3 days
tricyclic antidepressants <b>TCA</b>	1000	5-7 days
thc/marijuana <b>THC</b>	50	Infrequent use: 2-5 days Moderate use: 10-15 days Chronic use: 1 month

### TEST PROCEDURE AND RESULTS INTERPRETATION

Break the safety seal using corner of test card




Insert test card so urine level is within the blue area as indicated on the card.  
Leave undisturbed until control lines form and background clears.  
(Approx. 3-5 minutes)

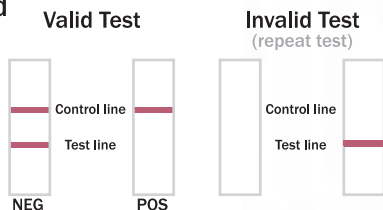

LINE INTENSITIES MAY VARY

control line = test valid  
no control line = test invalid  
test line = test negative  
no test line = test positive

This test provides only a preliminary qualitative test result. Use a more specific alternate quantitative analytical method to obtain a confirmed analytical result.

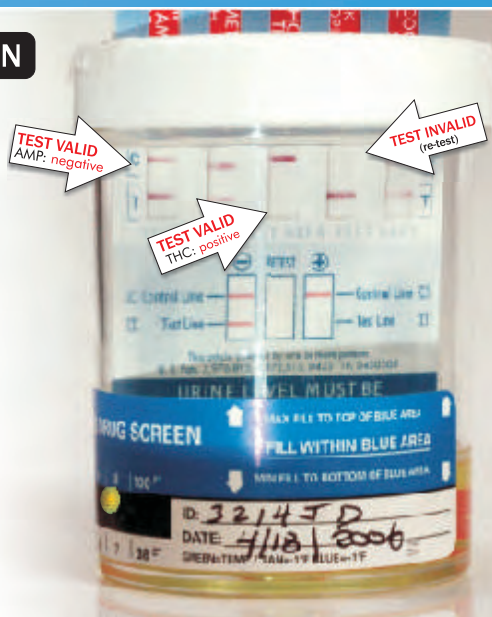

13-ADV-018-02

800.227.1243 ■ 518.758.8158

www.abmc.com