

# QUICK REFERENCE GUIDE

Rapid TOX Cup® II is a cost effective integrated collection and testing product capable of testing two to fourteen (2-14) drugs of abuse simultaneously.

STORE AT ROOM TEMPERATURE 59-86°F/15-30°C

TECHNICAL SUPPORT AVAILABLE 24/7/365  
800.227.1243 or 518.758.8158 ext. 3

## SPECIMEN COLLECTION

Drug name	Cut-off ng/mL	Detection Times (approximate)
<b>AMP</b> amphetamines	500 1000	2-4 days
<b>BAR</b> barbiturates	300	2-4 days
<b>BZO</b> benzodiazepines	300	Up to 2 weeks
<b>BUP</b> buprenorphine	12.5	2-4 days
<b>COC</b> cocaine	150 300	1-3 days
<b>MDMA</b> ecstasy	500 1000	2-4 days
<b>METH</b> methamphetamines	500 1000	1-2 days
<b>MTD</b> methadone	300	1-4 days
<b>OPI</b> opiates	300 2000	1-3 days
<b>OXY</b> oxycodone	100	1-3 days
<b>PCP</b> phencyclidine	25	3-8 days
<b>PPX</b> propoxyphene	300	1-3 days
<b>TCA</b> tricyclic antidepressants	1000	5-7 days
<b>THC</b> marijuana	50	Infrequent use: 2-5 days Moderate use: 10-15 days Chronic use: 1 month



### VERIFY EXPIRATION DATE

Lot and expiration date are embossed at the top of the pouch

### OPEN POUCH

Just prior to sample collection

### INSTRUCT DONOR TO PROVIDE SAMPLE

Sample level must be above the minimum fill line printed on the cup

### VERIFY SPECIMEN TEMPERATURE

LINE INTENSITIES MAY VARY

### GREEN DOT = TEMPERATURE

Read within 4 minutes  
Acceptable Specimen:  
90-100°F / 32-38°C

A visual check for specimen integrity may also be performed




## RESULT INTERPRETATION

The results may be interpreted once the control lines have formed and the background has cleared (typically this occurs in three to five (3-5) minutes).

Results are stable for up to 6 hours.


### Valid Test

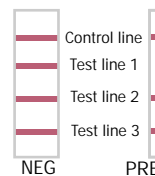

NEG

PRELIM POS  
(No line at test line 1)

### Invalid Test (repeat test)

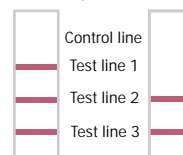

control line = test valid  
no control line = test invalid  
test line = test negative  
no test line = test preliminary positive

This test provides only a preliminary qualitative test result.  
Use a more specific alternate quantitative analytical method to obtain a confirmed analytical result.

Prior to using Rapid TOX Cup II read complete test procedure, found in this Quick Reference Guide and manufacturer's Product Instructions. The Rapid TOX Cup II test is designed for use with human urine specimens only.