

STORE AT ROOM TEMPERATURE 59-86°F/15-30°C

Rapid TOX Cup® II is a cost effective integrated collection and testing device capable of testing up to 14 drugs of abuse simultaneously.

**IMPORTANT NOTE:** When reading your results, a line appearing next to the drug abbreviation on the cup, indicates a **Negative Result**. No line appearing next to the drug abbreviation indicates a **Preliminary Positive** result.

## BEFORE YOU BEGIN TESTING


- Read these instructions in full.
- Check the lot # and expiration date on the front of the pouch or on the box. Do not use if product is expired.

TECHNICAL SUPPORT AVAILABLE 24/7/365  
800.227.1243 ext. 3

## STEP 1: SAMPLE COLLECTION

Remove test from foil pouch and remove lid from the testing cup.

Urinate into the cup, the sample level must be above the minimum fill line located on the bottom edge of the cups label.

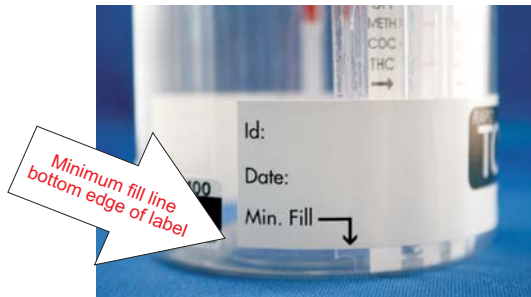

## STEP 2: CHECK SAMPLE TEMPERATURE

**GREEN DOT = TEMPERATURE**

Read within 4 minutes


Acceptable Sample Temperature:  
90-100°F / 32-38°C



## STEP 3: RESULT INTERPRETATION

Negative results may be interpreted once the control lines have formed and the background has cleared. Preliminary Positive results should not be recorded before the 5 minute mark.


**RESULTS ARE STABLE FOR UP TO 4 HOURS**

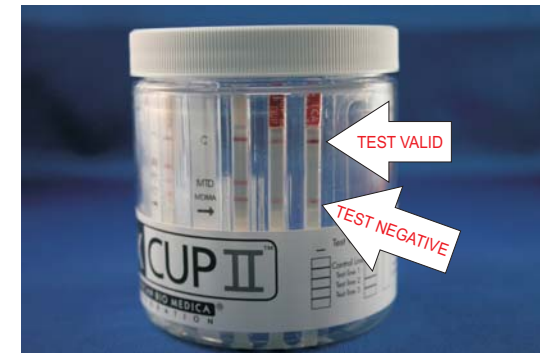


**LINE INTENSITIES MAY VARY**

**Control Line = Test Valid**  
**No Control Line = Test Invalid**  
**Test Line = Test Negative**  
**No Test Line = Test Preliminary Positive**

### Multi-drug Strip Interpretation

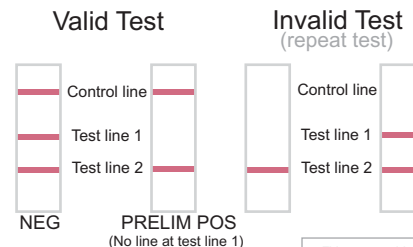

### Single Drug Strip Interpretation

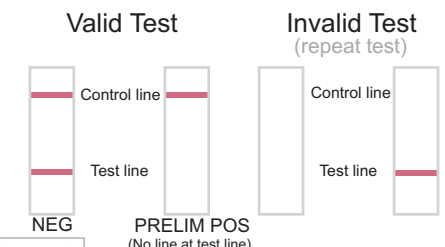

This test provides only a preliminary positive or negative test result. For more specific results please send the tested sample to a laboratory for confirmation testing.

Drug name	Cut-off ng/mL	Detection Times (approximate)
<b>AMP</b> amphetamines	500 1000	2-4 days
<b>BAR</b> barbiturates	300	2-4 days
<b>BZO</b> benzodiazepines	300	Up to 2 weeks
<b>BUP</b> buprenorphine	12.5	2-4 days
<b>COC</b> cocaine	150 300	1-3 days
<b>MDMA</b> ecstasy	500 1000	2-4 days
<b>METH</b> methamphetamines	500 1000	1-2 days
<b>MTD</b> methadone	300	1-4 days
<b>OPI</b> opiates	300 2000	1-3 days
<b>OXY</b> oxycodone	100	1-3 days
<b>PCP</b> phencyclidine	25	3-8 days
<b>PPX</b> propoxyphene	300	1-3 days
<b>TCA</b> tricyclic antidepressants	1000	5-7 days
<b>THC</b> marijuana	50	Infrequent use: 2-5 days Moderate use: 10-15 days Chronic use: 1 month

### 1. The Drug Line is lighter than the Control Line; what does this mean?

Any line next to the drug abbreviation is a line. Regardless of how dark or light the line may be, it is considered a **Negative Result** and no further testing is required. It is possible that the intensity of the lines will vary among the drugs being tested due to various reasons such as: how diluted the urine is, the pH or protein level of the urine, or an interference from a metabolite in the urine that closely resembles the drug being tested for. It is important to keep in mind that no matter how faint the drug line may appear, it is still considered a **Negative Result**.

### 2. What does Preliminary Positive mean?

The Rapid TOX Cup II drug test is a “screening test”. It is recommended that all **Preliminary Positive Results** be sent to the laboratory for confirmation testing. Gas chromatography/mass spectrometry (GC/MS) testing is the standard for laboratory testing of most drugs.

### 3. How do I know if the Rapid TOX Cup II worked correctly? What if I am uncertain of a result?

The test is working correctly if the reddish-purple control line has formed. The result is **Negative** when a reddish-purple line appears next to a drug abbreviation. The result is **Preliminary Positive** when NO line forms next to a drug abbreviation.

Please feel free to call our dedicated Technical Support team at 1-800-227-1243 ext.3 with any questions. They are available 24/7/365.

### 4. How soon can I read my test results?

**Negative Results** may be interpreted once the control line(s) have formed and the background has cleared. **Preliminary Positive** results should not be recorded before the 5 minute mark. All results are stable for up to 4 hours.

### 5. Are there any factors that can affect this test?

- Certain over-the-counter medications or prescription drugs may cross-react with specific drugs in the Rapid TOX Cup II and cause a **Preliminary Positive Result**. For more information or any questions on cross-reactions please call our Technical Support Team at 1-800-227-1243 ext.3.
- The test will only give accurate results on fresh human urine samples. Old or diluted urine samples may not be suitable for testing.
- Please remember that the Rapid TOX Cup II drug test are only as accurate as the urine sample being tested. Samples can easily be “adulterated” (i.e., tampered with or contaminated) using products commonly found in households such as cleaning solutions, bleach and other liquids. It is recommended that you closely supervise the process in its entirety.
- The Rapid TOX Cup II drug test provides a screening result only. It is not designed to determine the actual concentration of drug or the level of intoxication.

### QUESTIONS?

Call our toll-free line to reach technical support. Available  
24/7/365  
1-800-227-1243 ext.3